

If during the operation of your chair you either see visible smoke or smell smoke disconnect power from your chair immediately. You have 2 choices at this point. If you do not mind operating your chair WITHOUT seat or leg vibration proceed to step 1. If you must have seat and leg vibration please contact Premier Health Products to arrange the return of your chair for service.


Disconnect power from chair before proceeding!

Step 1) Refer to the set up instructions for you chair to remove the left armrest (left as you are sitting in the chair).

Step 2) Remove the 3mm allen screw holding the cover over the expansion board.



Step 3) Disconnect plugs from ports 6 & 7 on the expansion board.

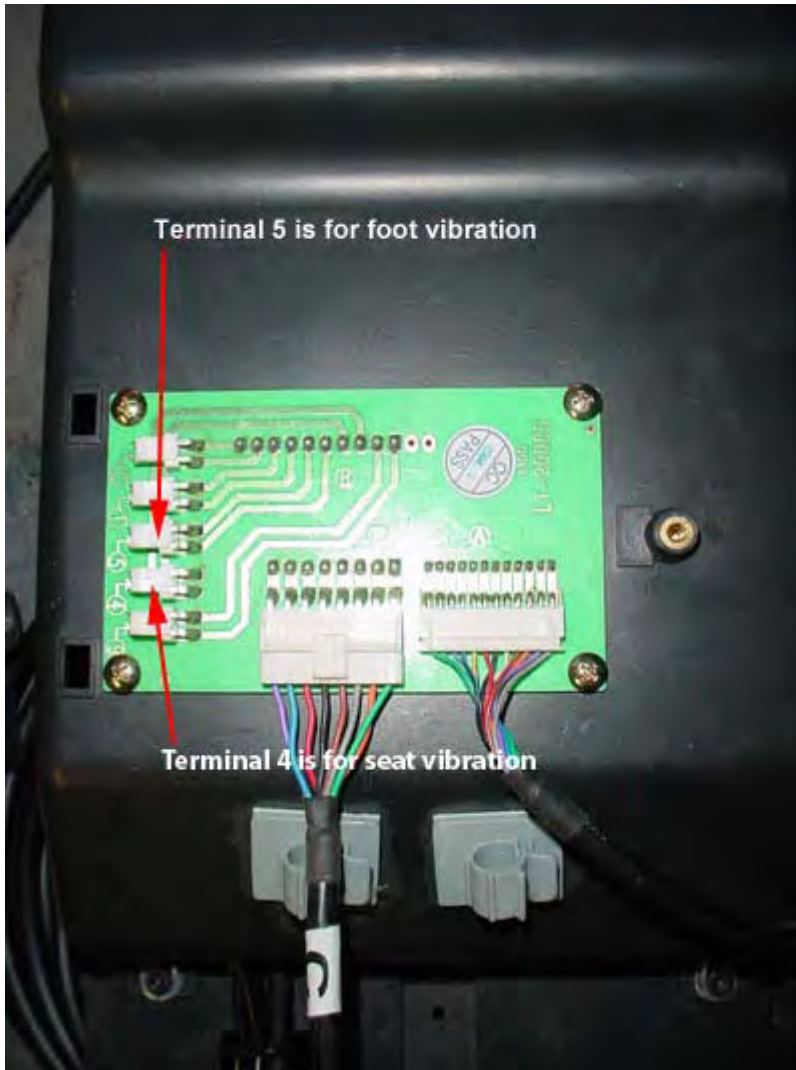

Step 4) Replace cover over expansion board, power up chair and verify all function less seat and leg vibration work properly. If chair does not operate properly please contact Premier Health Products to arrange return of your chair for service.