

## PHP 2030 Boxing Instructions

For questions, clarifications or assistance call 303-670-8882

TOOLS REQUIRED: 1) heavy duty shipping / packaging tape.

*BEFORE BEGINNING, ensure the power switch on the chair is in the off position and unplug the power cord .*  
Allow the chair to sit for 10 minutes before proceeding.

If you do not have all of the appropriate packing materials, use an appropriate substitute (i.e. use rolled-up bubble wrap in place of the foam blocks). The packaging of the chair to avoid shipping damage is the responsibility of the parties boxing / packaging the chair.

**Images are for instructional purposes, you WILL NOT need to remove your arms if your chair came in one box.**



Reach under the back of the chair with your right hand.....about 6” in from the left side and about 4” in from the back cover. Locate the latch mechanism with a finger and pull latch out to release.






Hold onto the seat back until it is firmly lying on the seat.  
*DO NOT ALLOW chair back to drop unsupported onto the seat.*



**CAUTION!** When the latch is released, chair back will be free weight and will pivot.  
Hold the top of the chair and lay it forward gently. Locate the rectangular leg and castor protectors, there will be 2

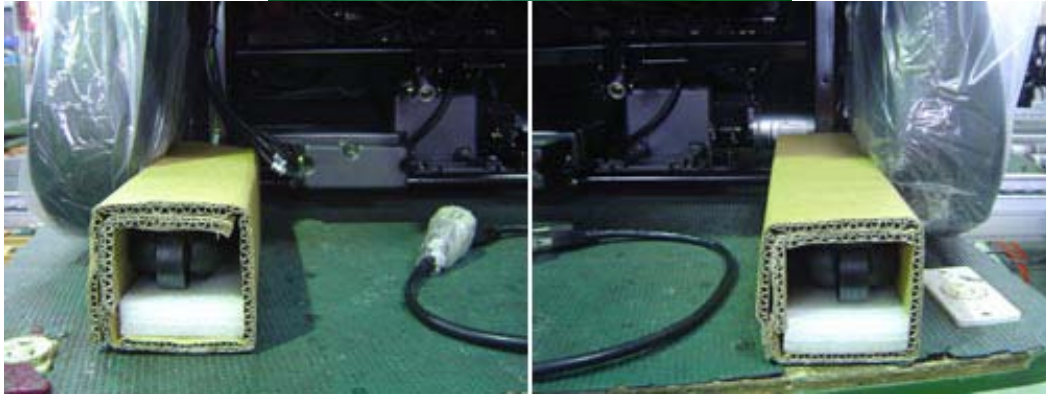

Place the protectors over the legs and castors as shown.



Wrap the chair in the provided plastic wrap, around the back, arms and footrest. Place the chair in the box as shown.



Place the back pad on top of the chair



Close the box and secure it with heavy duty packing tape.