

**PHP 2030 Back Upholstery Removal Instructions**  
For questions, clarifications or assistance call 303-670-8882

*Tools required*

One Phillips screwdriver, medium length

**BEFORE BEGINNING, turn the power switch on the chair OFF and unplug the power cord.**  
Allow the chair to sit for 10 minutes before proceeding.

**STEP 1**

Lay the back of the chair down over the seat of the chair. You can get a close up of the [back latch by clicking here.](#)



Reach under the back of the chair with your right hand.....about 6” in from the left side and about 4” in from the back cover. Locate the latch mechanism with a finger and pull latch out to release.





CAUTION, when the latch is release it will be free weight and will pivot.  
Hold the top of the chair and lay forward gently.



Hold onto the seat back until it is firmly lying on the seat.  
*DO NOT ALLOW chair back to 'drop' unsupported onto the seat.*  
**Step 2**




Once the back of the chair is folded down and you are ready to begin, locate the six screws holding the back cover on.



Using your # 2 Phillips screw driver remove the six screws.  
Lift the back cover up and off and set aside.

*For Model PHP2030, disconnect the air hose (indicated by arrow here) before removing the back cover.*

Remove the backrest upholstery by removing the six 5mm screws shown below.

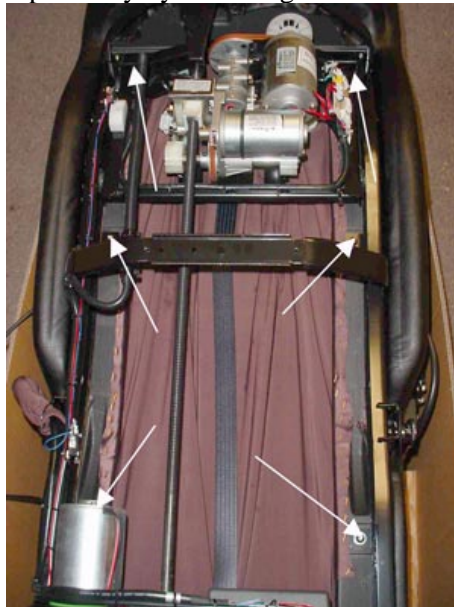

*If after reassembly, and during the final 5-minute test, you detect anything that seems inconsistent with normal operation of the chair, call Premier Health Products Customer Support at 303-670-8882 for immediate assistance.*